

**NEW 2024**  
with index

# STOMA CARE PRODUCTS AND SOLUTIONS

A complete guide to our comprehensive range of colostomy, ileostomy, urostomy and accessory products



Recognising Salts research into healthy stoma skin



DERMATOLOGICALLY ACCREDITED

**WESTECH**  
HEALTH CARE LTD.

 **salts**  
HEALTHCARE



One of the founders of ostomy care products, Salts Healthcare is a family-owned company that has been in business since 1701.

With unique and cutting-edge skin-friendly technology, they are the first ostomy care company to receive recognition by the British Skin Foundation for their research into hydrocolloids; and are also dermatologically accredited by the Skin Health Alliance.

By bringing family values into everything they do, they've earned a reputation for producing innovative and user-friendly products that improve lives.

Westech Health Care Ltd shares Salts Healthcare's values and is proud to distribute Salts Healthcare products in Canada.

# Contents

Sharing skincare credentials with the world's best brands ..... 4

## Closed

Confidence® Natural Advance..... 6

Confidence® Natural Soft Convex..... 8

Confidence® Convex Supersoft..... 10

Harmony® Duo with Flexifit® and Aloe ..... 12

## Drainable

Confidence® Natural Advance..... 14

Confidence® Natural Soft Convex..... 16

Confidence® Convex Supersoft..... 18

Confidence® Comfort Neonatal/Paediatric with Flexifit®..... 20

Confidence® Comfort..... 21

Harmony® Duo with Flexifit® and Aloe ..... 22

## Urostomy

Confidence® Natural Advance..... 24

Confidence® Convex Supersoft..... 26

Night Drainage Bag..... 26

Confidence® Wound and Fistula Pouch with Flexifit® (non-sterile) ..... 28

## Additional Products

Salts Mouldable Seals with Aloe..... 30

Salts Mouldable Seals..... 31

Salts Aloe Rings **NEW**..... 32

Salts Stoma Collar..... 33

Salts Flange Extender..... 34

Salts Adhesive Remover..... 35

Salts Barrier Film..... 36

Salts No-Sting Stoma Paste..... 37

Salts Stoma Paste ..... 37

Salts Odour Eliminating Spray..... 38

Salts Protective Wafers..... 39

Salts No-Roma..... 40

Salts Adjustable Ostomy Belt ..... 41

Salts Skin Lotion..... 42

Notes..... 43

# SHARING SKIN CREDENTIALS WITH THE WORLD'S BEST BRANDS



DERMATOLOGICALLY  
ACCREDITED



Recognising Salts research  
into healthy stoma skin

## Protecting skin. It's in our nature

Our research into skin-friendly hydrocolloids is one of the reasons we've achieved recognition from the Skin Health Alliance. The SHA is the 'leading international skin health authority, working closely with dermatologists, researchers and skin scientists' around the globe.

The SHA awards its dermatological accreditation mark to products and services where scientific and clinical research achieves, or surpasses, its stringent skin safety criteria.

**This accreditation puts Salts products among some of the leading skin-friendly brands in the world, such as Olay, Bio-Oil®, Carex, Neutrogena® and Pampers®.**

## Naturally better for skin

Salts Healthcare was the first, and is still the only, ostomy manufacturer to receive recognition from the British Skin Foundation for our skin-friendly hydrocolloids. As one of the UK's leading supporters of skin disease research, the BSF independently reviews companies to ensure their products are not detrimental to skin health.

**Its support puts us in the company of some of the world's most respected skincare brands, including Bio-Oil®, Boots, Garnier and sk:n, who, like Salts, also use Aloe Vera in their products.**





Every one of our best selling products\* featuring the Flexifit® wafer is infused with extracts of Aloe Vera, providing a natural way that may help to soothe and protect skin around the stoma.

Since the ancient Egyptians and Chinese first discovered this powerful plant thousands of years ago, it has become well known for its moisturising and healing properties.

The addition of Aloe Vera in our hydrocolloids is important as it acts as a humectant, which helps to moderate skin hydration.



\*Except Confidence® Comfort Flexifit® Neonatal and Paediatric wafers.

# CONFIDENCE<sup>®</sup> NATURAL ADVANCE

Confidence<sup>®</sup> Natural Advance features our new superior adhesion hydrocolloid that sticks faster, stronger and longer, for improved security and less chance of leaks. The unique five-sided Flexifit<sup>®</sup> wafer is designed to give the perfect fit. Plus, the wafer is infused with extracts of Aloe Vera, which may help to soothe and protect your skin. In addition, the new two-colour release film features a bold blue cutting guide, which makes it easier if you choose to cut your own bags.



## Features:

- The superior adhesion hydrocolloid has been developed to stick faster, stronger and for longer.
- Every Confidence<sup>®</sup> Natural Advance wafer is infused with extracts of Aloe Vera, providing a natural way that may help to soothe and protect the skin around your stoma.
- The unique five-sided Flexifit<sup>®</sup> wafer was developed to create the perfect fit, by bending and flexing with your body's natural contours to help reduce leaks and improve comfort.
- The two-colour release film, with bold blue cutting guide, makes it easier to cut your own bags.
- A bold blue release tab makes it easier to locate and remove the release film.
- Charcoal filter and internal film layer design help to minimise odour and pancaking by improving airflow.
- Soft beige cover with transparent overlap for easy viewing and positioning.



### LARGE CLOSED BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES		PACK SIZE
			Transparent	Trans/Overlap	
13–90mm	•	–	NALT1390**	NAL1390**	30
13mm	•	–	NALT13*	NAL13*	30

### STANDARD CLOSED BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES		PACK SIZE
			Transparent	Trans/Overlap	
13mm	•	–	NAT13*	NA13*	30
25mm	–	•	–	NA25	30
28mm	–	•	–	NA28	30
32mm	–	•	–	NA32	30
35mm	–	•	–	NA35	30
38mm	–	•	–	NA38	30
41mm	–	•	–	NA41	30

### MINI CLOSED BAG

SIZE	STARTER HOLE	PRE-CUT	ORDER CODE	PACK SIZE
			Trans/Overlap	
13mm	•	–	NAM13*	30


\*Starter hole wafers can be cut up to 70mm wide and 55mm high.

\*\*Large wafers can be cut up to 90mm wide and 80mm high.

# CONFIDENCE<sup>®</sup>

## NATURAL SOFT CONVEX

With our softest, stickiest and most flexible soft convex wafer, Confidence<sup>®</sup> Natural Soft Convex has a range of special features designed to help soft convex wearers enjoy a better fit, a better feel and better quality of life, including a unique five-sided Flexifit<sup>®</sup> wafer to give the perfect fit. Plus, the wafer is infused with extracts of Aloe Vera, which may help to soothe and protect your skin.

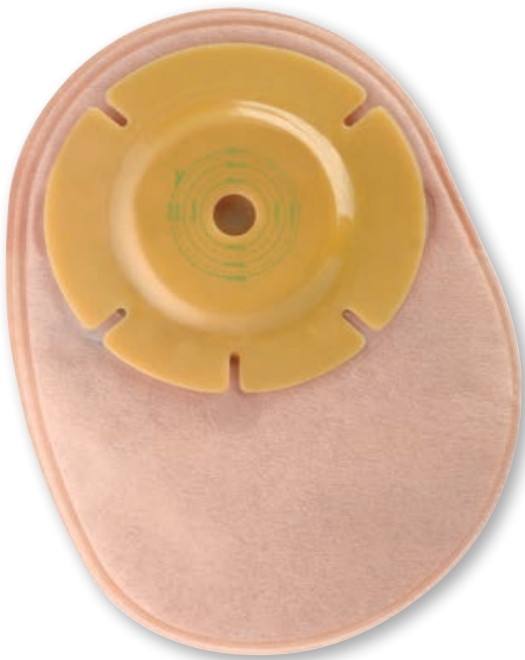

### Features:

- The wafer's strong adhesive has been developed to stick fast and stay put for longer, improving security and helping reduce leaks. It features an incredibly soft convex insert, making Confidence<sup>®</sup> Natural Soft Convex incredibly flexible and comfortable.
- The unique five-sided Flexifit<sup>®</sup> wafer was developed to create the perfect fit, by bending and flexing with your body's natural contours to help reduce leaks and improve comfort.
- Every Confidence<sup>®</sup> Natural wafer is infused with extracts of Aloe Vera, providing a natural way that may help to soothe and protect the skin around your stoma.
- Soft belt loops, for extra security and comfort.
- Charcoal filter and internal film layer design help to minimise odour and pancaking by improving airflow.
- Soft beige cover with transparent overlap, for easy viewing and positioning.





### LARGE CLOSED BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES Trans/Overlap	PACK SIZE
13–25mm	•	–	XNL1325	10
13–38mm	•	–	XNL1338	10
13–52mm	•	–	XNL1352	10

### STANDARD CLOSED BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES Trans/Overlap	PACK SIZE
13–25mm	•	–	XN1325	10
13–38mm	•	–	XN1338	10
13–52mm	•	–	XN1352	10
21mm	–	•	XN21	10
25mm	–	•	XN25	10
28mm	–	•	XN28	10
32mm	–	•	XN32	10
35mm	–	•	XN35	10
38mm	–	•	XN38	10
41mm	–	•	XN41	10

### MINI CLOSED BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES Trans/Overlap	PACK SIZE
13–25mm	•	–	XNM1325	10
13–38mm	•	–	XNM1338	10

Note: this product is specifically designed for colostomists with flush or retracted stomas, or where the stoma is situated in skin folds; and should only be used after prior assessment by an ET Nurse.

# CONFIDENCE<sup>®</sup> CONVEX SUPERSOFT

For those who require it, Confidence<sup>®</sup> Convex Supersoft offers a slightly firmer level of convexity than our popular Confidence<sup>®</sup> Natural Soft Convex product.



## Features:

- Comfortable and flexible wafer with gentle convex insert.
- Belt loops for extra security.
- Soft beige cover with transparent overlap, for easy viewing and positioning.



### LARGE CLOSED BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES Trans/Overlap	PACK SIZE
13–25mm	•	–	CCSSL1325	10
13–38mm	•	–	CCSSL1338	10
13–52mm	•	–	CCSSL1352	10

### STANDARD CLOSED BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES Trans/Overlap	PACK SIZE
13–25mm	•	–	CCSS1325	10
13–38mm	•	–	CCSS1338	10
13–52mm	•	–	CCSS1352	10
21mm	–	•	CCSS21	10
25mm	–	•	CCSS25	10
28mm	–	•	CCSS28	10
32mm	–	•	CCSS32	10
35mm	–	•	CCSS35	10
38mm	–	•	CCSS38	10
41mm	–	•	CCSS41	10

Note: this product is specifically designed for colostomists with flush or retracted stomas, or where the stoma is situated in skin folds; and should only be used after prior assessment by an ET Nurse.

# HARMONY<sup>®</sup> DUO WITH FLEXIFIT<sup>®</sup> AND ALOE

Harmony<sup>®</sup> Duo with Flexifit<sup>®</sup> and Aloe is a popular choice for those who enjoy the convenience of a two-piece system with the discretion of a one-piece. The stick-together range includes our unique Flexifit<sup>®</sup> wafer which is infused with extracts of Aloe Vera.

## STANDARD WAFER

with Flexifit<sup>®</sup> and  
Aloe extracts



## CONVEX WAFER

with Flexifit<sup>®</sup> and  
Aloe extracts



## Wafer features:

- The unique five-sided Flexifit<sup>®</sup> wafer was developed to create the perfect fit, by bending and flexing with your body's natural contours to help reduce leaks and improve comfort.
- Every Harmony<sup>®</sup> Duo wafer is infused with extracts of Aloe Vera, providing a natural way that may help to soothe and protect the skin around your stoma.
- Easy-to-apply stick-together system with locator which matches up with a notch on the bag for accurate positioning.
- Convex wafer available with comfortable and gentle convex insert and belt loops.

## Bag features:

- Notch on the bag matches up with the locator on the wafer, for an easy-to-apply, stick-together system.
- Soft beige opaque cover.



## STANDARD WAFERS – with Flexifit® and Aloe extracts

SIZE	ORDER CODES	PACK SIZE
13–32mm	FHD1332*	10
13–50mm	FHD1350*	10
13–70mm	FHD1370*	10

## CONVEX WAFERS – with Flexifit® and Aloe Vera extracts

SIZE	ORDER CODES	TO FIT BAGS			PACK SIZE	SIZE	ORDER CODES	TO FIT BAGS			PACK SIZE
		1332	1350	1370				1332	1350	1370	
13–25mm	XFHD1325*	✓	–	–	5	21mm	XFHD21†	✓	–	–	5
13–38mm	XFHD1338*	–	✓	–	5	25mm	XFHD25†	✓	–	–	5
13–52mm	XFHD1352*	–	–	✓	5	28mm	XFHD28†	✓	–	–	5
*Starter hole. †Pre-cut to fit 13–32mm bags. ††Pre-cut to fit 13–50mm bags. It doesn't matter if you need a small, standard or large bag, this matrix still applies.						32mm	XFHD32††	–	✓	–	5
						35mm	XFHD35††	–	✓	–	5
						38mm	XFHD38††	–	✓	–	5

### LARGE CLOSED BAGS

SIZE	ORDER CODES Opaque	PACK SIZE
13–32mm	HDCL1332	30
13–50mm	HDCL1350	30
13–70mm	HDCL1370	30

### STANDARD CLOSED BAGS

SIZE	ORDER CODES Opaque	PACK SIZE
13–32mm	HDC1332	30
13–50mm	HDC1350	30
13–70mm	HDC1370	30

Note: this product is specifically designed for colostomists with flush or retracted stomas, or where the stoma is situated in skin folds; and should only be used after prior assessment by an ET Nurse.

# CONFIDENCE<sup>®</sup> NATURAL ADVANCE

Confidence<sup>®</sup> Natural Advance features our new superior adhesion hydrocolloid that sticks faster, stronger and longer, for improved security and less chance of leaks. The unique five-sided Flexifit<sup>®</sup> wafer is designed to give the perfect fit. Plus, the wafer is infused with extracts of Aloe Vera, which may help to soothe and protect your skin. In addition, the new two-colour release film features a bold blue cutting guide, which makes it easier if you choose to cut your own bags.



## Features:

- The superior adhesion hydrocolloid has been developed to stick faster, stronger and for longer.
- Every Confidence<sup>®</sup> Natural Advance wafer is infused with extracts of Aloe Vera, providing a natural way that may help to soothe and protect the skin around your stoma.
- The unique five-sided Flexifit<sup>®</sup> wafer was developed to create the perfect fit, by bending and flexing with your body's natural contours to help reduce leaks and improve comfort.
- The two-colour release film, with bold blue cutting guide, makes it easier to cut your own bags.
- A bold blue release tab makes it easier to locate and remove the release film.
- The easy-to-use outlet features a discreet opaque film and drainage tab that helps to open and empty the bag.
- Discreet closure tucks up under the soft cover.
- Charcoal filter and internal film layer design help to minimise odour and ballooning by improving airflow.
- Soft beige cover with transparent overlap for easy viewing and positioning.



### LARGE DRAINABLE BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES		PACK SIZE
			Transparent	Trans/Overlap	
13–90mm	•	–	NDALT1390**	NDAL1390**	30
13mm	•	–	NDALT13*	NDAL13*	30

### STANDARD DRAINABLE BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES		PACK SIZE
			Transparent	Trans/Overlap	
13mm	•	–	NDAT13*	NDA13*	30
25mm	–	•	–	NDA25	30
28mm	–	•	–	NDA28	30
32mm	–	•	–	NDA32	30
35mm	–	•	–	NDA35	30

### SMALL DRAINABLE BAG

SIZE	STARTER HOLE	PRE-CUT	ORDER CODE	PACK SIZE
			Trans/Overlap	
13mm	•	–	NDAS13*	30

\*Starter hole wafers can be cut up to 70mm wide and 55mm high.

\*\*Large wafers can be cut up to 90mm wide and 80mm high.

# CONFIDENCE<sup>®</sup>

## NATURAL SOFT CONVEX

With our softest, stickiest and most flexible soft convex wafer, Confidence<sup>®</sup> Natural Soft Convex has a range of special features designed to help soft convex wearers enjoy a better fit, a better feel and better quality of life, including a unique five-sided Flexifit<sup>®</sup> wafer to give the perfect fit. Plus, the wafer is infused with extracts of Aloe Vera, which may help to soothe and protect your skin.



### Features:

- The wafer's strong adhesive has been developed to stick fast and stay put for longer, improving security and helping reduce leaks. It features an incredibly soft convex insert, making Confidence<sup>®</sup> Natural Soft Convex incredibly flexible and comfortable.
- The unique five-sided Flexifit<sup>®</sup> wafer was developed to create the perfect fit, by bending and flexing with your body's natural contours to help reduce leaks and improve comfort.
- Every Confidence<sup>®</sup> Natural wafer is infused with extracts of Aloe Vera, providing a natural way that may help to soothe and protect the skin around your stoma.
- Soft belt loops, for extra security and comfort.
- The easy-to-use outlet features a discreet opaque film and drainage tab that helps you to open and empty the bag.
- Discreet closure tucks up under the soft cover.
- Charcoal filter and internal film layer design help to minimise odour and ballooning by improving airflow.
- Soft beige cover with transparent overlap, for easy viewing and positioning.





### LARGE DRAINABLE BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES Trans/Overlap	PACK SIZE
13–25mm	•	–	XNDL1325	10
13–38mm	•	–	XNDL1338	10
13–52mm	•	–	XNDL1352	10

### STANDARD DRAINABLE BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES Trans/Overlap	PACK SIZE
13–25mm	•	–	XND1325	10
13–38mm	•	–	XND1338	10
13–52mm	•	–	XND1352	10
21mm	–	•	XND21	10
25mm	–	•	XND25	10
28mm	–	•	XND28	10
32mm	–	•	XND32	10
35mm	–	•	XND35	10
38mm	–	•	XND38	10
41mm	–	•	XND41	10

### SMALL DRAINABLE BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES Trans/Overlap	PACK SIZE
13–25mm	•	–	XNDS1325	10
13–38mm	•	–	XNDS1338	10

Note: this product is specifically designed for colostomists with flush or retracted stomas, or where the stoma is situated in skin folds; and should only be used after prior assessment by an ET Nurse.

# CONFIDENCE<sup>®</sup> CONVEX SUPERSOFT

For those who require it, Confidence<sup>®</sup> Convex Supersoft offers a slightly firmer level of convexity than our popular Confidence<sup>®</sup> Natural Soft Convex product.



## Features:

- Comfortable and flexible wafer with gentle convex insert.
- Belt loops for extra security.
- The outlet rolls up to a secure closure with hook-and-loop wings.
- Discreet closure tucks up under the soft cover and is securely held in place by hidden hook material, for extra comfort and security.
- Soft beige cover with transparent overlap, for easy viewing and positioning.



### LARGE DRAINABLE BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES Trans/Overlap	PACK SIZE
13–25mm	•	–	CDSSL1325	10
13–38mm	•	–	CDSSL1338	10
13–52mm	•	–	CDSSL1352	10

### STANDARD DRAINABLE BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES Trans/Overlap	PACK SIZE
13–25mm	•	–	CDSS1325	10
13–38mm	•	–	CDSS1338	10
13–52mm	•	–	CDSS1352	10
21mm	–	•	CDSS21	10
25mm	–	•	CDSS25	10
28mm	–	•	CDSS28	10
32mm	–	•	CDSS32	10
35mm	–	•	CDSS35	10
38mm	–	•	CDSS38	10
41mm	–	•	CDSS41	10

### SMALL DRAINABLE BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES Trans/Overlap	PACK SIZE
13–25mm	•	–	CDSSS1325	10
13–38mm	•	–	CDSSS1338	10
13–52mm	•	–	CDSSS1352	10

Note: Confidence® Convex Supersoft should only be used after prior assessment by an ET Nurse.

# CONFIDENCE<sup>®</sup> COMFORT NEONATAL/ PAEDIATRIC WITH FLEXIFIT<sup>®</sup>



Confidence<sup>®</sup> Comfort Neonatal/Paediatric with Flexifit<sup>®</sup> is the perfect choice for tiny babies, toddlers or older children. Available in Neonatal or Paediatric sizes, it features a unique 6-sided Flexifit<sup>®</sup> wafer designed to fit snugly to a child's natural contours. It stays in place, helping to reduce leaks and irritation, even as they wriggle and move around. To help protect baby's delicate young skin, there's a skin-friendly hydrocolloid that is gentle on little bodies.

## Features:

- Available in Neonatal or Paediatric sizes.
- The unique six-sided Flexifit<sup>®</sup> wafer bends and flexes with the child or baby's movements, and the oval shape allows drainage to the side if the child is lying down.
- 'No hole' options available to cut your own shaped hole into the wafer for flexible positioning.
- Soft and easy-to-clean outlet rolls up to secure closure with hook-and-loop wings.
- Soft beige transparent overlap for easy viewing and positioning or transparent options available.



## PAEDIATRIC DRAINABLE BAGS

SIZE	ORDER CODES	Trans/Overlap	PACK SIZE
8–40mm Starter Hole	CFP8		30
NO HOLE	CFPNH		30

## NEONATAL DRAINABLE BAGS

SIZE	ORDER CODES		PACK SIZE
	Transparent	Trans/Overlap	
8–40mm Starter Hole	CFNT8	CFN8	30
NO HOLE	CFNTNH	CFNNH	30

# CONFIDENCE<sup>®</sup> COMFORT



Two bags, each with a large oval wafer – perfect for large stomas or extra security. Wafers can be cut up to 110mm wide and 70mm top-to-bottom.

### Features:

- Oval shaped wafer – perfect for loop stomas.
- The outlet rolls up to secure closure with hook-and-loop wings.
- Beige cover with transparent overlap for easy viewing and positioning.



LARGE DRAINABLE BAG				
SIZE	STARTER HOLE	PRE-CUT	ORDER CODE Trans/Overlap	PACK SIZE
13mm	•	–	DLTO13*	30
STANDARD DRAINABLE BAG				
SIZE	STARTER HOLE	PRE-CUT	ORDER CODE Trans/Overlap	PACK SIZE
13mm	•	–	DTO13*	30

\*Oval wafer bags can be cut up to 110mm wide and 70mm high

# HARMONY<sup>®</sup> DUO WITH FLEXIFIT<sup>®</sup> AND ALOE

Harmony<sup>®</sup> Duo with Flexifit<sup>®</sup> and Aloe is a popular choice for those who enjoy the convenience of a two-piece system with the discretion of a one-piece. The stick-together range includes our unique Flexifit<sup>®</sup> wafer which is infused with extracts of Aloe Vera.

## STANDARD WAFER

with Flexifit<sup>®</sup> and  
Aloe extracts



## CONVEX WAFER

with Flexifit<sup>®</sup> and  
Aloe extracts



## Wafer features:

- The unique five-sided Flexifit<sup>®</sup> wafer was developed to create the perfect fit, by bending and flexing with your body's natural contours to help reduce leaks and improve comfort.
- Every Harmony<sup>®</sup> Duo wafer is infused with extracts of Aloe Vera, providing a natural way that may help to soothe and protect the skin around your stoma.
- Easy-to-apply stick-together system with locator which matches up with a notch on the bag for accurate positioning.
- Convex wafer available with comfortable and gentle convex insert and belt loops.

## Bag features:

- Notch on the bag matches up with the locator on the wafer, for an easy-to-apply, stick-together system.
- The outlet rolls up to secure closure with hook-and loop wings.
- Discreet closure tucks up under the soft cover and is securely held in place by hidden hook material, for extra comfort and security.
- Soft beige opaque cover with transparent overlap for easy viewing and positioning.



## STANDARD WAFERS – with Flexifit® and Aloe extracts

SIZE	ORDER CODES	PACK SIZE
13–32mm	FHD1332*	10
13–50mm	FHD1350*	10
13–70mm	FHD1370*	10

## CONVEX WAFERS – with Flexifit® and Aloe extracts

SIZE	ORDER CODES	TO FIT BAGS	PACK SIZE	SIZE	ORDER CODES	TO FIT BAGS	PACK SIZE
		1332 1350 1370				1332 1350 1370	
13–25mm	XFHD1325*	✓ – –	5	21mm	XFHD21†	✓ – –	5
13–38mm	XFHD1338*	– ✓ –	5	25mm	XFHD25†	✓ – –	5
13–52mm	XFHD1352*	– – ✓	5	28mm	XFHD28†	✓ – –	5
*Starter hole. †Pre-cut to fit 13–32mm bags. ††Pre-cut to fit 13–50mm bags. It doesn't matter if you need a small, standard or large bag, this matrix still applies.				32mm	XFHD32††	– ✓ –	5
				35mm	XFHD35††	– ✓ –	5
				38mm	XFHD38††	– ✓ –	5

## LARGE DRAINABLE BAGS

SIZE	ORDER CODES	PACK SIZE
	Transparent      Trans/Overlap	
13–32mm	–      HDDL1332	30
13–50mm	HDDL1350      HDDL1350	30
13–70mm	HDDL1370      HDDL1370	30

## STANDARD DRAINABLE BAGS

## SMALL DRAINABLE BAGS

SIZE	ORDER CODES	PACK SIZE	SIZE	ORDER CODES	PACK SIZE
	Trans/Overlap			Trans/Overlap	
13–32mm	HDD1332	30	13–32mm	HDDS1332	30
13–50mm	HDD1350	30	13–50mm	HDDS1350	30
13–70mm	HDD1370	30	13–70mm	HDDS1370	30

Note: this product is specifically designed for colostomists with flush or retracted stomas, or where the stoma is situated in skin folds; and should only be used after prior assessment by an ET Nurse.

# CONFIDENCE<sup>®</sup> NATURAL ADVANCE

Confidence<sup>®</sup> Natural Advance features our new superior adhesion hydrocolloid that sticks faster, stronger and longer, for improved security and less chance of leaks. The unique five-sided Flexifit<sup>®</sup> wafer is designed to give the perfect fit. Plus, the wafer is infused with extracts of Aloe Vera, which may help to soothe and protect your skin. In addition, the new two-colour release film features a bold blue cutting guide, which makes it easier if you choose to cut your own bags.



## Features:

- The superior adhesion hydrocolloid is a urostomy-specific formulation that has been developed to stick faster, stronger and for longer by resisting large amounts of fluid and improving wear-time.
- Every Confidence<sup>®</sup> Natural Advance wafer is infused with extracts of Aloe Vera, providing a natural way that may help to soothe and protect the skin around your stoma.
- The unique five-sided Flexifit<sup>®</sup> wafer was developed to create the perfect fit, by bending and flexing with your body's natural contours to help reduce leaks and improve comfort.
- The two-colour release film, with bold blue cutting guide, makes it easier to cut your own bags.
- A bold blue release tab makes it easier to locate and remove the release film.
- The hard tap with easy to use 'on/off' opening is ideal if you have poor dexterity.
- Soft beige cover with transparent overlap for easy viewing and positioning.





### STANDARD UROSTOMY BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES Trans/Overlap	PACK SIZE
13mm	•	–	NUA13*	10
25mm	–	•	NUA25	10
28mm	–	•	NUA28	10
32mm	–	•	NUA32	10
35mm	–	•	NUA35	10

### SMALL UROSTOMY BAG

SIZE	STARTER HOLE	PRE-CUT	ORDER CODE Trans/Overlap	PACK SIZE
13mm	•	–	NUAS13*	10

\*Starter hole wafers can be cut up to 70mm wide and 55mm high.

# CONFIDENCE<sup>®</sup> CONVEX SUPERSOFT

For those who require it, Confidence<sup>®</sup> Convex Supersoft offers a slightly firmer level of convexity than our popular Confidence<sup>®</sup> Natural Soft Convex product.



## Salts Night Drainage Bag

Salts Night Drainage Bag has a large 2,000ml capacity and a 1,200mm tube to give you greater movement, for a comfortable night's rest. And, with a unique swivel action and quick snap-on connection, Salts Night Drainage Bag is even easier for you to use.





## STANDARD UROSTOMY BAGS

SIZE	STARTER HOLE	PRE-CUT	ORDER CODES Trans/Overlap	PACK SIZE
13–25mm	•	–	CUSS1325	10
13–38mm	•	–	CUSS1338	10
13–52mm	•	–	CUSS1352	10
21mm	–	•	CUSS21	10
25mm	–	•	CUSS25	10
28mm	–	•	CUSS28	10
32mm	–	•	CUSS32	10
35mm	–	•	CUSS35	10
38mm	–	•	CUSS38	10

Note: this product is specifically designed for colostomists with flush or retracted stomas, or where the stoma is situated in skin folds; and should only be used after prior assessment by an ET Nurse.

## SALTS NIGHT DRAINAGE BAG

SIZE	ORDER CODE	PACK SIZE	DESCRIPTION	ORDER CODE	PACK SIZE
2,000ml	ZL0400	10	Adaptor	NDA6	6

# CONFIDENCE<sup>®</sup> WOUND AND FISTULA POUCH with FLEXIFIT<sup>®</sup>

(non-sterile)

A unique wound management product with many innovative features. Our small bag is available with horizontal or vertical wafers, depending on the wound or fistula position. Our medium bag has a long, narrow-shaped wafer to fit perfectly between other pouches or dressings – ideal for open (non-sterile) mid-line incisional wounds. All products benefit from our unique Flexifit<sup>®</sup> wafer, which helps to achieve the perfect fit.



## Features:

- This 1-piece drainable product, available in three different sizes, is suitable for a variety of fistulas and wounds.
- The unique Flexifit<sup>®</sup> wafer is only 1mm thick. Its six slits create flexible compression zones for a closer fit, which is ideal for challenging, uneven surfaces.
- The woundcare adhesive has been specifically formulated to resist breakdown.
- Our bags can cater for low to medium exudate and come with odour-resistant films.
- We only use skin-friendly hydrocolloids, research into which is recognised by the British Skin Foundation, and dermatologically accredited by the Skin Health Alliance.
- The non-return valve and wide-bore spout outlet allows even thick exudate to drain.
- Continuous drainage can be attached internally or externally to the outlet spout.



## CONFIDENCE® WOUND AND FISTULA POUCH with FLEXIFIT® (non-sterile)

SIZE	ORDER CODES	PACK SIZE
<b>SMALL</b> – Horizontal wounds up to 60mm wide and 40mm high	CWM01	10
<b>SMALL</b> – Vertical wounds up to 40mm wide and 60mm high	CWM02	10
<b>MEDIUM</b> – Vertical wounds up to 53mm wide and 150mm high	CWM03	10

# SALTS MOULDABLE SEALS WITH ALOE



Salts Mouldable Seals with Aloe are extremely mouldable seals. They help to prevent leaks that cause sore skin by moulding to the exact shape and size of your stoma. Following the contours of the abdomen to help create a customised, comfortable, leak-resistant seal, Salts Mouldable Seals with Aloe are infused with extracts of Aloe Vera to create a skin-friendly seal that may help protect the skin from redness and irritation.



## SALTS MOULDABLE SEALS WITH ALOE

DIAMETER	THICKNESS	ORDER CODES	PACK SIZE
15mm – 40mm	3mm	SMSAT15	30
35mm – 70mm	3mm	SMSAT35	30
15mm – 40mm	4.2mm	SMSAS15	30
35mm – 70mm	4.2mm	SMSAS35	30
15mm – 100mm	3mm	SMSAL	10

# SALTS MOULDABLE SEALS



Salts Mouldable Seals are extremely mouldable rings that help to prevent leaks by moulding to the exact shape and size of your stoma. They follow the contours of the abdomen to help create a customised, comfortable and leak-resistant seal. The skin-friendly formulation helps to protect the skin from redness and irritation. By helping to extend wear-time, Salts Mouldable Seals are ideal for people living with an ileostomy or urostomy.



SALTS MOULDABLE SEALS				
SIZE	DIAMETER	THICKNESS	ORDER CODES	PACK SIZE
Thin	50mm	3mm	SMST	30
Standard	50mm	4.2mm	SMSS	30
Large	100mm	3mm	SMSL	10

# SALTS ALOE RINGS



Salts Aloe Rings are skin-friendly, thin and flexible hydrocolloid rings. They can be stretched to accommodate moderately oval stomas and retain their shape for a secure, comfortable fit. Infused with extracts of Aloe Vera, Salts Aloe Rings may help to soothe, moisturise and protect peristomal skin.



SALTS ALOE RINGS		
INTERNAL DIAMETER	ORDER CODES	PACK SIZE
25mm	SAR25	30
32mm	SAR32	30
38mm	SAR38	30
45mm	SAR45	30



# SALTS STOMA COLLAR



This innovative stoma collar provides a leak-proof barrier around the base of the stoma. The protective collar can help to give your stoma direction and more shape, resulting in a better spout that points directly into the bag, helping to prevent output from coming into contact with the skin. Ideal for all round-shaped stoma types, the thin and flexible collar is made of an extremely soft material that twists and turns with the movement of your body.



## SALTS STOMA COLLAR

STOMA SIZE	ORDER CODES	PACK SIZE
17mm – 20mm	DC20	30
21mm – 23mm	DC23	30
24mm – 26mm	DC26	30
27mm – 29mm	DC29	30
30mm – 32mm	DC32	30
33mm – 35mm	DC35	30
36mm – 38mm	DC38	30
39mm – 41mm	DC41	30

# SALTS FLANGE EXTENDER



When experiencing difficulties with the adhesion of your bag, Salts Flange Extender with Aloe can provide a simple and effective solution. These thin, skin-friendly hydrocolloid strips are easy to apply and help secure the edges of your wafer, giving you added security wherever you need it. And, with added extracts of Aloe Vera, Salts Flange Extender with Aloe may help to protect the delicate skin around your stoma. Also available in a non-Aloe Vera option.



## SALTS FLANGE EXTENDER WITH ALOE VERA

SIZE	ORDER CODE	PACK SIZE
One size fits all	SPHA2	30

## SALTS FLANGE EXTENDER

SIZE	ORDER CODE	PACK SIZE
One size fits all	SPH1	30

# SALTS ADHESIVE REMOVER



Salts Adhesive Remover effectively removes residue and refreshes, helping to leave skin clean and adhesive-free. Containing a non-sting formula with a natural peppermint fragrance, Salts Adhesive Remover helps to mask odour, leaving you feeling fresh after every change. Available in a spray or wipe.



## SALTS ADHESIVE REMOVER

DESCRIPTION	ORDER CODE	QUANTITY
50ml	WAPX	1
Wipes	WA1	30 sachets

# SALTS BARRIER FILM

Leakage and trauma caused by the constant changing of a bag can lead to sore skin. Salts Barrier Film is specifically designed to leave a gentle, protective film on the skin immediately around the stoma. It can help to protect the skin against leakages, prevent sore skin due to frequent bag changes, and perspiration penetrating the adhesive. Apply to clean skin and allow to dry before bag application. Available in a wipe or foam applicator.



## SALTS BARRIER FILM

DESCRIPTION	ORDER CODES	PACK SIZE
Wipes	PPS1	30 sachets
Foam Applicator	PPS2	5

# SALTS NO-STING STOMA PASTE




Salts No-Sting Stoma Paste helps to provide an ideal surface for your bag to adhere to, by filling any dips or creases to help prevent leaks. And, as it's alcohol-free, Salts skin-friendly No-Sting Stoma Paste has none of the stinging associated with traditional pastes.



SALTS NO-STING STOMA PASTE		
SIZE	ORDER CODE	QUANTITY
60g	NSP1	1

# SALTS STOMA PASTE

Stoma Paste has been specifically designed to be used as an effective filler where there is scar tissue or gaps around the stoma. Stoma Paste is easy to use and, once applied, can be spread into position, providing an ideal surface for appliances to adhere to.

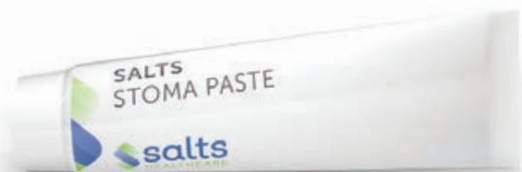

SALTS STOMA PASTE		
SIZE	ORDER CODE	QUANTITY
60g	SP60	1

# SALTS ODOUR ELIMINATING SPRAY



Unlike many other air fresheners and deodorisers, Salts Odour Eliminating Spray doesn't just mask unpleasant smells with perfume, but attacks and destroys the odour molecules completely, so the odour can't come back.

- Eliminates odour – it really does work!
- Odours are eliminated at a molecular level.
- Not just another masking or neutralising spray.
- Non-toxic, non-hazardous and non-flammable.
- Environmentally friendly.
- Non-allergenic.

Salts Odour Eliminating Spray is effective and easy to use. During pouch changes, Salts Odour Eliminating Spray is simply sprayed once or twice into the air around the pouch.

- Lightweight and a handy size.
- Can be used safely in confined spaces.
- Will not harm skin surfaces or fabrics.
- A hint of peppermint.



## SALTS ODOUR ELIMINATING SPRAY

SIZE	ORDER CODE	QUANTITY
50ml	FA1	1

# SALTS PROTECTIVE WAFERS



An allergic or undesirable reaction to an adhesive used on a particular appliance may be the cause of sore skin.

Indeed, the adhesive itself may not even be strong enough to fully secure the appliance.

- Non-irritating and suitable for sensitive skin.
- Thinner than other leading brands, for maximum discretion.
- Easy to cut and simple to apply.
- Promotes healing when used on broken skin.
- Flexible and soft, for added comfort.

Protective Wafers are specifically designed to help prevent soreness and increase adhesion. Salts' hydrocolloid wafers provide an ideal environment to promote healing of sore or broken skin – and then help prevent the condition from reoccurring.



## SALTS PROTECTIVE WAFERS

SIZE	ORDER CODES	QUANTITY
10cm x 10cm	PW1010	10
15cm x 15cm	PW1515	5

# SALTS NO-ROMA

Unpleasant odour escaping from an appliance can be very embarrassing. No-Roma, the easy-to-use pouch deodorant, is specifically designed to prevent this situation from occurring.

- Kills the odour-producing bacteria.
- Available in two sizes – pocket size is ideal to carry around.
- Completely odourless.
- Lasting results.

Unlike other products, No-Roma works by actually killing the bacteria that produce the odour, rather than just simply masking it with perfume. It is also very simple to use. When changing an appliance, a drop of No-Roma is placed inside the bag (not on the stoma itself) to prevent the build-up of odour.

No-Roma can be used with any appliance, and is available in a small bottle, making it ideal to discreetly carry around.



SALTS NO-ROMA			
SIZE	ORDER CODES	QUANTITY	
28ml	833021	1	
227ml	833056	1	



# SALTS

## ADJUSTABLE OSTOMY BELT

To complement our range of additional products, we offer a Salts Adjustable Ostomy Belt. Compatible with the majority of two-piece and convex bags, Salts Adjustable Ostomy Belt provides added security for your peace of mind.



### SALTS ADJUSTABLE OSTOMY BELT

SIZE	ORDER CODES	QUANTITY
100cm	AB01	1
120cm	AB02	1

# SALTS SKIN LOTION



Skin irritation and soreness can be uncomfortable and distressing. Salts Skin Lotion doesn't contain alcohol, so it can be used on sore skin without stinging. Quick and easy-to-use, simply apply it to a dry wipe and dab onto the affected area. It isn't greasy and helps the bag to adhere better to the skin when fully dry. Suitable for use around any stoma type, under any type of appliance.



SALTS SKIN LOTION			
DESCRIPTION	SIZE	ORDER CODE	QUANTITY
Lotion	28ml	833006SAMP	1

# SALTS SUPPORT WEAR

Available in black, white and nude, our ladies' briefs and unisex boxer are made from a two-way stretch fabric that allows the body to move freely, while ribbing on the front provides a flush finish, making them virtually invisible under most clothing.



SALTS SUPPORT WEAR – UNISEX BOXER			SALTS SUPPORT WEAR – LADIES' BRIEFS		
SIZE	COLOUR	ORDER CODES	SIZE	COLOUR	ORDER CODES
X Small (22–27")	Black	BOXBXS	XS (4/6/8)	Black	BRFBXS
X Small (22–27")	White	BOXWXS	XS (4/6/8)	White	BRFWXS
X Small (22–27")	Nude	BOXNXS	XS (4/6/8)	Nude	BRFNXS
Small/Med (28/30/32")	Black	BOXBSM	S/M (8/10/12)	Black	BRFBSM
Small/Med (28/30/32")	White	BOXWSM	S/M (8/10/12)	White	BRFWSM
Small/Med (28/30/32")	Nude	BOXNSM	S/M (8/10/12)	Nude	BRFNSM
Med/Large (34/36/38")	Black	BOXBML	M/L (14/16/18)	Black	BRFBML
Med/Large (34/36/38")	White	BOXWML	M/L (14/16/18)	White	BRFWML
Med/Large (34/36/38")	Nude	BOXNML	M/L (14/16/18)	Nude	BRFNML
Large/X Large (40/42/44")	Black	BOXBLXL	L/XL (20/22/24)	Black	BRFBLXL
Large/XLarge (40/42/44")	White	BOXWLXL	L/XL (20/22/24)	White	BRFWLXL
Large/XLarge (40/42/44")	Nude	BOXNLXL	L/XL (20/22/24)	Nude	BRFNLXL
XXLarge (46/48/50")	Black	BOXBXXL	XXL (26/28/30)	Black	BRFBXXL
XX Large (46/48/50")	White	BOXWXXL	XXL (26/28/30)	White	BRFWXXL
XXLarge (46/48/50")	Nude	BOXNXXL	XXL (26/28/30)	Nude	BRFNXXL

# INDEX

<i>One Piece Closed</i>		<i>Page #</i>
NAL13	Confidence Natural Advanc Large O/Lap Closed 13mm Starter Hole	.....07
NAL1390	Confidence Natural Advanc Large O/Lap Closed 13-90mm Starter Hole	.....07
NALT13	Confidence Natural Advanc Large Transp Closed 13mm Starter Hole	.....07
NALT1390	Confidence Natural Advanc Large Transp Closed 13-90mm Starter Hole	.....07
NAT13	Confidence Natural Advanc Std Transp Closed 13mm Starter Hole	.....07
NAM13	Confidence Natural Advanc Mini O/Lap Closed 13mm Starter Hole	.....07
NA13	Confidence Natural Advanc Std O/Lap Closed 13mm Starter Hole	.....07
NA25	Std O/Lap Closed Conf Nat Adv Pre-Cut 25mm	.....07
NA28	Confidence Natural Advanc Std O/Lap Closed 28mm Pre-Cut	.....07
NA32	Confidence Natural Advanc Std O/Lap Closed 32mm Pre-Cut	.....07
NA35	Confidence Natural Advanc Std O/Lap Closed 35mm Pre-Cut	.....07
NA38	Confidence Natural Advanc Std O/Lap Closed 38mm Pre-Cut	.....07
NA41	Confidence Natural Advanc Std O/Lap Closed 41mm Pre-Cut	.....07
XN1325	Natural Convex Std 13-25 Cut To Fit Trans/Olap Closed	.....07
XN1338	Natural Convex Std 13-38 Cut To Fit Trans/Olap Closed	.....09
XN1352	Natural Convex Std 13-52 Cut To Fit Trans/Olap Closed	.....09
XN21	Natural Convex Std 21mm Pre-Cut Trans/Olap Closed	.....09
XN25	Natural Convex Std 25mm Pre-Cut Trans/Olap Closed	.....09
XN28	Natural Convex Std 28mm Pre-Cut Trans/Olap Closed	.....09
XN32	Natural Convex Std 32mm Pre-Cut Trans/Olap Closed	.....09
XN35	Natural Convex Std 35mm Pre-Cut Trans/Olap Closed	.....09
XN38	Natural Convex Std 38mm Pre-Cut Trans/Olap Closed	.....09
XN41	Natural Convex Std 41mm Pre-Cut Trans/Olap Closed	.....09
XNL1325	Natural Convex Lge 13-25 Cut To Fit Trans/Olap Closed	.....09
XNL1338	Natural Convex Lge 13-38 Cut To Fit Trans/Olap Closed	.....09
XNL1352	Natural Convex Lge 13-52 Cut To Fit Trans/Olap Closed	.....09
XNM1325	Natural Convex Sml 13-25 Cut To Fit Trans/Olap Closed	.....09
XNM1338	Natural Convex Sml 13-38 Cut To Fit Trans/Olap Closed	.....09
CCSS1325	Supersoft Convex Cl Cut To Fit 13-25mm Overlap Non Woven	.....11
CCSS1338	Supersoft Convex Cl Cut To Fit 13-38mm Overlap Non Woven	.....11

# INDEX

<b>One Piece Closed</b>		<b>Page #</b>
CCSS1352	Supersoft Convex Cl Cut To Fit 13-52mm Overlap Non Woven	.....11
CCSS21	Supersoft Convex Cl Pre Cut 21mm Hole Overlap Non Woven	.....11
CCSS25	Supersoft Convex Cl Pre Cut 25mm Hole Overlap Non Woven	.....11
CCSS28	Supersoft Convex Cl Pre Cut 28mm Hole Overlap Non Woven	.....11
CCSS32	Supersoft Convex Cl Pre Cut 32mm Hole Overlap Non Woven	.....11
CCSS35	Supersoft Convex Cl Pre Cut 35mm Hole Overlap Non Woven	.....11
CCSS38	Supersoft Convex Cl Pre Cut 38mm Hole Overlap Non Woven	.....11
CCSS41	Clsd Convex Pouch Uk Pre Cut 41mm Hole Overlap Non Woven	.....11
CCSSL1325	Ssoft Convex Cl Lge Cut To Fit 13-25mm Overlap Non Woven	.....11
CCSSL1338	Ssoft Convex Cl Lge Cut To Fit 13-38mm Overlap Non Woven	.....11
CCSSL1352	Ssoft Convex Cl Lge Cut To Fit 13-52mm Overlap Non Woven	.....11
<b>One Piece Drainable</b>		
CDSS1325	Supersoft Convex Dr 13-25 Cut To Fit Hole N/Woven Filter	.....19
CDSS1338	Supersoft Convex Dr 13-38 Cut To Fit Hole N/Woven Filter	.....19
CDSS1352	Supersoft Convex Dr 13-52 Cut To Fit Hole N/Woven Filter	.....19
CDSS21	Supersoft Convex Dr Pouch 21mm Hole N/Woven Filter	.....19
CDSS25	Supersoft Convex Dr Pouch 25mm Hole N/Woven Filter	.....19
CDSS28	Supersoft Convex Dr Pouch 28mm Hole N/Woven Filter	.....19
CDSS32	Supersoft Convex Dr Pouch 32mm Hole N/Woven Filter	.....19
CDSS35	Supersoft Convex Dr Pouch 35mm Hole N/Woven Filter	.....19
CDSS38	Supersoft Convex Dr Pouch 38mm Hole N/Woven Filter	.....19
CDSS41	Supersoft Convex Dr Pouch 41mm Hole N/Woven Filter	.....19
CDSSL1325	Ssoft Convex Dr Lge 13-25 Cut To Fit Hole N/Woven Filter	.....19
CDSSL1338	Ssoft Convex Dr Lge 13-38 Cut To Fit Hole N/Woven Filter	.....19
CDSSL1352	Ssoft Convex Dr Lge 13-52 Cut To Fit Hole N/Woven Filter	.....19
CDSSS1325	Ssoft Convex Dr Sml 13-25 Cut To Fit Hole N/Woven Filter	.....19
CDSSS1338	Ssoft Convex Dr Sml 13-38 Cut To Fit Hole N/Woven Filter	.....19
CDSSS1352	Ssoft Convex Dr Sml 13-52 Cut To Fit Hole N/Woven Filter	.....19
CFN8	Confcomf. Flexi Neonatal Overlap 8mm Starter Hole	.....20
CFNNH	Confcomf. Flexi Neonatal Overlap No Hole	.....20

# INDEX

<b>One Piece Drainable</b>		<b>Page #</b>
CFP8	Confcomf. Flexi Paed Overlap 8mm Starter Hole	.....20
CFPNH	Confcomf. Flexi Paed Overlap No Hole	.....20
DLTO13	Conf Comf Large Oval With O/L Large Pouch 13mm Starter Hole	.....21
DTO13	Conf Comf Drain Oval With O/L Std Pouch 13mm Starter Hole	.....21
NDA13	Confidence Nat Advance Drainable Std Olap 13mm Starter Hole	.....15
NDA25	Confidence Nat Advance Drainable Std Olap 25mm Pre Cut	.....15
NDA28	Confidence Nat Advance Drainable Std Olap 28mm Pre Cut	.....15
NDA32	Confidence Nat Advance Drainable Std Olap 32mm Pre Cut	.....15
NDA35	Confidence Nat Advance Drainable Std Olap Pre-Cut 35mm	.....15
NDAL13	Confidence Nat Advance Drainable Lge Olap 13mm Starter Hole	.....15
NDAL1390	Large O/Lap Drainable 13-90mm Starter Hole Confidence Natural Adva	.....15
NDALT13	Large Transp Drainable 13mm Starter Hole Confidence Natural Adva With Superior Adhesion Hydrocolloid	.....15
NDALT1390	Large Transp Drainable 13-90mm Starter Hole Confidence Natural Adva	.....15
NDAS13	Confidence Nat Advance Drainable Sml Olap 13-70mm	.....15
NDAT13	Std Transp Drainable 13mm Starter Hole Confidence Natural Adva	.....15
XND1325	Natural Convex Std 13-25 Cut To Fit Trans/Olap Drainable	.....17
XND1338	Natural Convex Std 13-38 Cut To Fit Trans/Olap Drainable	.....17
XND1352	Natural Convex Std 13-52 Cut To Fit Trans/Olap Drainable	.....17
XND21	Natural Convex Std 21mm Precut Trans/Overlap Drainable	.....17
XND25	Natural Convex Std 25mm Precut Trans/Overlap Drainable	.....17
XND28	Natural Convex Std 28mm Precut Trans/Overlap Drainable	.....17
XND32	Natural Convex Std 32mm Precut Trans/Overlap Drainable	.....17
XND35	Natural Convex Std 35mm Precut Trans/Overlap Drainable	.....17
XND38	Natural Convex Std 38mm Precut Trans/Overlap Drainable	.....17
XND41	Natural Convex Std 41mm Precut Trans/Overlap Drainable	.....17
XNDL1325	Natural Convex Lge 13-25 Cut To Fit Trans/Olap Drainable	.....17
XNDL1338	Natural Convex Lge 13-38 Cut To Fit Trans/Olap Drainable	.....17
XNDL1352	Natural Convex Lge 13-52 Cut To Fit Trans/Olap Drainable	.....17

# INDEX

<b>One Piece Drainable</b>		<b>Page #</b>
XNDS1325	Natural Convex Sml 13-25 Cut To Fit Trans/Olap Drainable	.....17
XNDS1338	Natural Convex Sml 13-38 Cut To Fit Trans/Olap Drainable	.....17
<b>One Piece Urostomy</b>		
CUSS1325	Cvex Urost S.soft Cut To Fit 13 - 25mm Transparent/Overlap	.....27
CUSS1338	Cvex Urost S.soft Cut To Fit 13 - 38mm Transparent/Overlap	.....27
CUSS1352	Cvex Urost S.soft Cut To Fit 13 - 52mm Transparent/Overlap	.....27
CUSS21	Cvex Urost S.soft Pre Cut 21mm Hole Transparent/Overlap	.....27
CUSS25	Cvex Urost S.soft Pre Cut 25mm Hole Transparent/Overlap	.....27
CUSS28	Cvex Urost S.soft Pre Cut 28mm Hole Transparent/Overlap	.....27
CUSS32	Cvex Urost S.soft Pre Cut 32mm Hole Transparent/Overlap	.....27
CUSS35	Cvex Urost S.soft Pre Cut 35mm Hole Transparent/Overlap	.....27
CUSS38	Cvex Urost S.soft Pre Cut 38mm Hole Transparent/Overlap	.....27
NUA13	Std O/Lap Urostomy 13mm Starter Hole Confidence Natural Advance	.....25
NUA25	Std O/Lap Urostomy 25mmpre Cut Confidence Natural Advanc	.....25
NUA28	Std O/Lap Urostomy 28mm Pre Cut Confidence Natural Advanc	.....25
NUA32	Std O/Lap Urostomy 32mm Starter Hole Confidence Natural Advanc	.....25
NUA35	Std O/Lap Urostomy 35mm Pre Cut Confidence Natural Advanc	.....25
NUAS13	Small O/Lap Urostomy 13mm Starter Hole Confidence Natural Advanc	.....25
<b>Two Piece Closed (Pouches)</b>		
HDC1332	Harmony Duo 13 - 32 Closed Opaque Starter 13mm - 32mm	.....13
HDC1350	Harmony Duo 13 - 50 Closed Opaque Starter 13mm - 50mm	.....13
HDC1370	Harmony Duo 13 - 70 Closed Opaque Starter 13mm - 70mm	.....13
HDCL1332	Harmony Duo Large Closed Opaque Starter 13mm - 32mm	.....13
HDCL1350	Harmony Duo Large Closed Opaque Starter 13mm - 50mm	.....13
HDCL1370	Harmony Duo Large Closed Opaque Starter 13mm - 70mm	.....13
<b>Two Piece Drainable (Pouches)</b>		
HDD1332	Harmony Duo Standard Drainable Opaque Starter 13mm - 32mm	.....23
HDD1350	Harmony Duo Standard Drainable Opaque Starter 13mm - 50mm	.....23
HDD1370	Harmony Duo Standard Drainable Opaque Starter 13mm - 70mm	.....23
HDDL1332	Harmony Duo Large Drainable Opaque Starter 13mm - 32mm	.....23

# INDEX

<b><i>Two Piece Drainable (Pouches)</i></b>		<b><i>Page #</i></b>
HDDL1350	Harmony Duo Large Drainable Opaque Starter 13mm - 50mm	.....23
HDDL1370	Harmony Duo Large Drainable Opaque Starter 13mm - 70mm	.....23
HDDL1350	Harmony Duo Large Drainable Transparent Starter 13mm - 50mm	.....23
HDDL1370	Harmony Duo Large Drainable Transparent Starter 13mm - 70mm	.....23
HDDS1332	Harmony Duo Small Drainable Opaque Starter 13mm - 32mm	.....23
HDDS1350	Harmony Duo Small Drainable Opaque Starter 13mm - 50mm	.....23
HDDS1370	Harmony Duo Small Drainable Opaque Starter 13mm - 70mm	.....23
<b><i>Two Piece Flange</i></b>		
FHD1332	Harmony Duo Flexifit Standard Flange With Aloe 13mm - 32mm	.....13,23
FHD1350	Harmony Duo Flexifit Standard Flange With Aloe 13mm - 50mm	.....13,23
FHD1370	Harmony Duo Flexifit Standard Flange With Aloe 13mm - 70mm	.....13,23
FHD21	Harmony Duo Flexifit Standard Flange With Aloe 21mm	.....13,23
FHD25	Harmony Duo Flexifit Standard Flange With Aloe 25mm	.....13,23
FHD28	Harmony Duo Flexifit Standard Flange With Aloe 28mm	.....13,23
FHD32	Harmony Duo Flexifit Standard Flange With Aloe 32mm	.....13,23
FHD35	Harmony Duo Flexifit Standard Flange With Aloe 35mm	.....13,23
FHD38	Harmony Duo Flexifit Standard Flange With Aloe 38mm	.....13,23
FHDF1350	Harmony Duo Flexifit Surround Flange With Aloe 13mm - 50mm	.....13,23
FHDF1370	Harmony Duo Flexifit Surround Flange With Aloe 13mm - 70mm	.....13,23
FHDF21	Harmony Duo Flexifit Surround Flange With Aloe 21mm	.....13,23
FHDF25	Harmony Duo Flexifit Surround Flange With Aloe 25mm	.....13,23
FHDF28	Harmony Duo Flexifit Surround Flange With Aloe 28mm	.....13,23
FHDF32	Harmony Duo Flexifit Surround Flange With Aloe 32mm	.....13,23
FHDF35	Harmony Duo Flexifit Surround Flange With Aloe 35mm	.....13,23
XFHD1325	Harmny Duo Flexi Conv Flange With Aloe Aloe 13-25mm Cut To Fit	.....13,23
XFHD1338	Harmny Duo Flexi Conv Flange With Aloe Aloe 13-38mm Cut To Fit	.....13,23
XFHD1352	Harmny Duo Flexi Conv Flange With Aloe Aloe 13-52mm Cut To Fit	.....13,23
XFHD21	Harmny Duo Flexi Conv Flange With Aloe Aloe 21mm Pre Cut	.....13,23



# INDEX

<b>Two Piece Flange</b>		<b>Page #</b>
XFHD25	Harmny Duo Flexi Conv Flange With Aloe Aloe 25mm Pre Cut	.....13,23
XFHD28	Harmny Duo Flexi Conv Flange With Aloe Aloe 28mm Pre Cut	.....13,23
XFHD32	Harmny Duo Flexi Conv Flange With Aloe Aloe 32mm Pre Cut	.....13,23
XFHD35	Harmny Duo Flexi Conv Flange With Aloe Aloe 35mm Pre Cut	.....13,23
XFHD38	Harmny Duo Flexi Conv Flange With Aloe Aloe 38mm Pre Cut	.....13,23
<b>Flange Extenders</b>		
SPH1	Salts Flange Extender	.....34
SPHA2INT	Salts Secuplast Aloe Hydro Aloe Vera Export	.....34
<b>General Seals</b>		
PW1010	Protective Wafer 10cm X 10cm	.....39
PW1515	Protective Wafer 15cm X 15cm	.....39
<b>Mouldable Seals</b>		
SMSAL	Salts Mouldable Seals With Aloe Large	.....30
SMSAS15	Salts Mouldable Seals With Aloe Standard 15-40mm	.....30
SMSAS35	Salts Mouldable Seals With Aloe Standard 35-70mm	.....30
SMSAT15	Salts Mouldable Seals With Aloe Thin 15-40mm	.....30
SMSAT35	Salts Mouldable Seals With Aloe Thin 35-70mm	.....30
SMSL	Salts Mouldable Seals Large (Prev Secuplast) 100mm Od 3.0mm Thickness	.....31
SMSS	Salts Mouldable Seals Standard. (Secuplast) 50mm Od 4.2mm Thickness	.....31
SMST	Salts Mouldable Seals Thin.(Prev Secuplast) 50mm Od 3.0mm Thickness	.....31
<b>Night Drainage Bags</b>		
ZL0400	Salts Night Drainage Bags 2lt Non Sterile [Box Of 10]	.....27
NDA6	Night Drainage Adapt	.....27
<b>Salts Aloe Rings</b>		
SAR 25	Salts Aloe Ring 25mm, 1.8mm Thick	.....32
SAR 32	Salts Aloe Ring 32mm, 1.8mm Thick	.....32
SAR 38	Salts Aloe Ring 38mm, 1.8mm Thick	.....32
SAR 45	Salts Aloe Ring 45mm, 1.8mm Thick	.....32

# INDEX

<b><i>Skin Fillers &amp; Protectives</i></b>		<b><i>Page #</i></b>
833006SAMP	Saltair Lotion	.....42
NSP1	Salts No-Sting Stoma Paste. (Previously Secupaste)	.....37
PPS1	Salts Barrier Film Wipes (Peri-Prep Sensitive) (30 Sachets Per Box)	.....36
PPS2	Salts Barrier Film Foam Applicators (Periprep Sens)	.....36
SP60	Salts Stoma Paste	.....37
<b><i>Stoma Collars</i></b>		
DC20	Salts Stoma Collar 20mm (Previously Dermacol)	.....33
DC23	Salts Stoma Collar 23mm (Previously Dermacol)	.....33
DC26	Salts Stoma Collar 26mm (Previously Dermacol)	.....33
DC29	Salts Stoma Collar 29mm (Previously Dermacol)	.....33
DC32	Salts Stoma Collar 32mm (Previously Dermacol)	.....33
DC35	Salts Stoma Collar 35mm (Previously Dermacol)	.....33
DC38	Salts Stoma Collar 38mm (Previously Dermacol)	.....33
DC41	Salts Stoma Collar 41mm (Previously Dermacol)	.....33
<b><i>Support Wear (UNISEX BOXER)</i></b>		
BOXBXS	X Small (22–27") Black	.....43
BOXBSM	Small/Med (28/30/32") Black	.....43
BOXBML	Med/Large (34/36/38") Black	.....43
BOXBLXL	Large/X Large (40/42/44")	.....43
BOXBXXL	Xxlarge (46/48/50") Black	.....43
<b><i>Support Wear (LADIES' BRIEFS)</i></b>		
BRFBXS	Xs (4/6/8) Black	.....43
BRFBSM	S/M (8/10/12) Black	.....43
BRFBML	M/L (14/16/18) Black	.....43
BRFBLXL	L/XL (20/22/24) Black	.....43
BRFBXXL	Xxl (26/28/30) Black	.....43

# NOTES

---

---

---

---

---

---

---

---

---

---

## QUICK LINKS TO RESOURCES



Go to the Westech  
Health Care Webpage  
for more information



Salts Discovery Series  
Podcast

Contact us for free samples and details of the complete range of ostomy products available from Salts:

**Westech Health Care Ltd.**



**Toll free 1 844.323.0022**



**info@westechhealth.com**



**www.westechhealth.com**



**@SaltsHealthcare**



**SaltsHealthcare**



**SaltsHealthcare**



To hear stories about living with a stoma, listen and subscribe to our podcast **'Me, my bag and I'**, which is available free on iTunes and Spotify.

RM765793 12/21



® Registered trade marks of Salts Healthcare Ltd. ©Salts Healthcare Ltd 2021. Products and trade marks of Salts Healthcare Ltd are protected by UK and foreign patents, registered designs and trade marks. For further details, please visit [www.salts.co.uk](http://www.salts.co.uk)