




Accessibility to bathrooms and showers in just a few hours

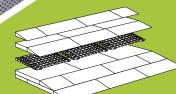



Improve everyday life









BATHROOM SOLUTIONS

Bathroom challenges

A wheelchair or a commode chair can cause many problems. Even the lowest brim may make rolling difficult and thus it requires heavy lifting from the nursing staff. Excellent ramps and tiles provide better accessibility and can be fitted to any bathroom.

Solutions with Excellent ramps and tiles are fast and economic and do not require any conversion of the bathroom.





Example of a typical shower unit. A shower tray like this one will be filled up with tiles and if necessary fitted with a small ramp if necessary.



Example of a typical shower unit. A shower tray like this one will be filled up with tiles and if necessary fitted with a small ramp.



BATHROOM SOLUTIONS

Threshold solutions

Access to the bathroom and the other rooms of a house is a necessity, and for this, we offer a range of solutions.

We can provide ramps not only in various heights and widths, with and without corner, but also in a number of different colours.

For more information see our brochure "Threshold Ramps" or www.ex-as.com.









Materials

All products are manufactured from the environmental friendly material PE (Polyethylene) and are:

100% Non-toxic 100% PVC-free 100% Reusable 100% Odourless

PE- Products are aciduric and non-corrosic.






SHOWER SOLUTIONS


Shower unit with lowered floor

Lowered floors in showers are fitted with Excellent tiles and possibly ramps. Hereby, a plane surface is achieved, and heavy lifting can be avoided.

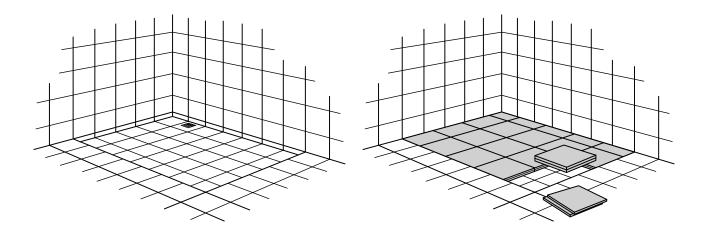

Shower trays with a brim

If the shower tray has a brim it is fitted with tiles in the tray and a ramp up to the edge.

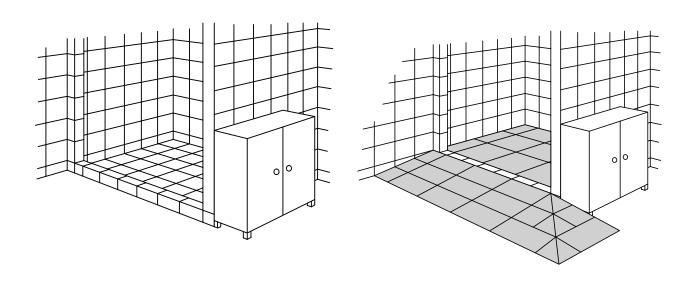




SHOWER SOLUTIONS


Shower niches with a low tray

The lowered shower tray is filled up with tiles and possibly ramps. A ramp is mounted at the edge.

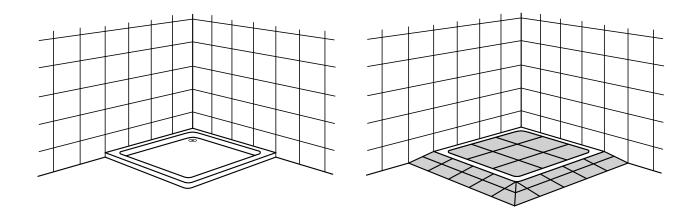

Lifting of the floor level


Finding room for a ramp in small bathrooms may be difficult. In these cases, the complete floor is lifted by filling it with tiles up to the edge of the shower tray and thus a flat surface is achieved.

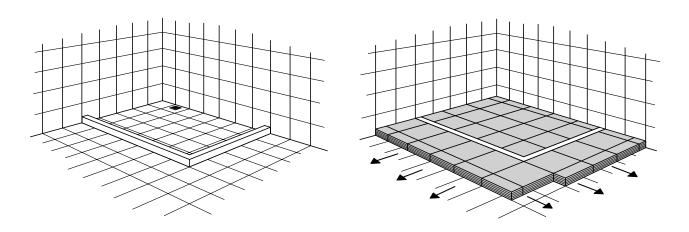




FITTING


Fitting of tiles in a shower unit:

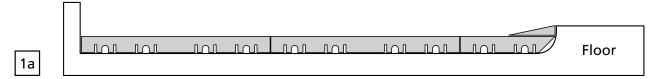

If the shower niche brim is higher than one tile and lower than two tiles, the tile floor can be mounted as a low floor in the shower tray.

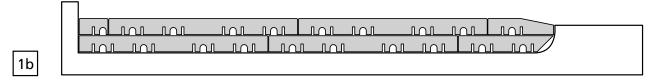

The tile floor in the shower tray can also be fitted as a higher floor. A ramp fitting the floor level is cut and mounted.

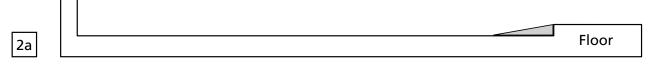

If the brim is lower than a tile, one may instead fit a small ramp down into the shower tray.




A tile floor may also be mounted higher than the brim, as shown here.

Fitting

2b

Any bathroom can be fitted 100 percent with Excellent tiles. The fitting can be done, e.g., with an ordinary jigsaw.

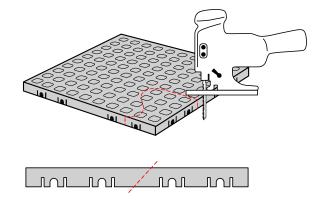




CLEANING OF TILES

The surface of the tiles and ramps has no pores and thus cannot absorb bacteria.

Water and soap can easily drain off through the holes in the tiles.

The tiles tolerate all types of conventional cleaning agents.

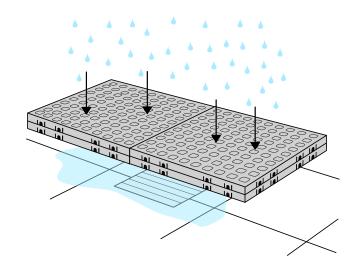







The tiles in the shower can easily be lifted and rinsed. Each layer of the tiles is removed and rinsed thoroughly. Finally, the floor of the shower is cleaned.



For even more thorough cleaning the tiles can be put into a dishwasher and washed with a disinfection and descaling program.

They also tolerate autoclaving (+130° C).



WESTECH HEALTH CARE LTD.

Westech Health Care Ltd.
154 - 5255 McCall Way NE, Calgary, Alberta T2E 7J5
2-8400 Parkhill Drive, Milton, Ontario L9T5V7

www.westechhealth.com :: orders@westechhealth.com :: sales@westechhealth.com :: 1-844-323-0022

Visit www.ramp-kits.com for videos and information.

